

Activation of a different hardware (pressure plate) in the BalanceLab software




Be sure to have the software key for the new hardware available!



Into BalanceLab connection bay (on the bottom side of the pressure plate)

Into Notebook /PC

1. Connect the USB cable of the new hardware to the PC/Laptop using the USB cable and be sure the power supply is connected.

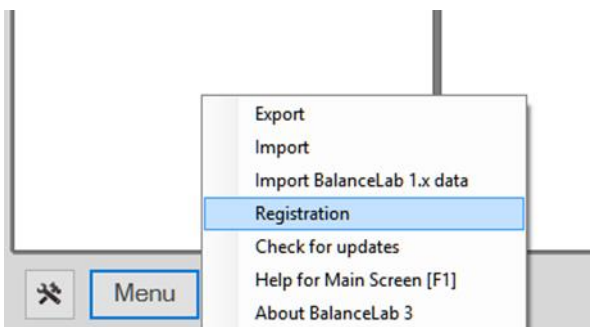

2. Start the SAM BalanceLab software
3. Click the "Menu" button in the main menu and select "Registration".

Activation of a different hardware (pressure plate) in the BalanceLab software

Registration Info

Currently active software registration

Active Software Key: vefo-15mu-iqez-7r6q

Program Edition: Pro

Assigned Hardware S/N: 00550912

Available registrations

Active	Hardware S/N	Software Key	Edition
X	00550912	vefo-15mu-iqez-7r6q	Pro

current connected hardware is marked blue / active registration is marked green

Add registration Remove registration Set as active registration

Currently installed hardware

Re-detect connected hardware

Hardware S/N: 1243005-00120703

Status: no software key available for connected hardware!

Close

☞ In the “Registration Info” window, you see all registered software keys/hardware. In the most cases, this would just be one hardware.

☞ In the box “Currently installed hardware”, you should now see your new hardware serial number (if the new system is connected successfully).

If this is the case, please proceed with Step 4, if not press “Re-Detect connected hardware” or check the USB connection status of the hardware.

4. Press the “Add registration” button.

SAM BalanceLab 3 - Enter Software Key

Please enter your software key to activate the product.

The key should have been provided with your system. If you cannot find your key please contact: support@scienceandmotion.com

Hardware S/N number

00120703

Software Key input

v7ob - 1535 - ittt - 7r6q

OK Cancel

☞ A new window will open with input fields for the 16-digit software key.

5. Please enter the new key.




Be careful not to mix up numbers and letters. Especially number “1” and letter “l” (“L”) or number “0” (zero) and letter “o” (as Otto) can easily be misinterpreted depending on the font used to print the software key.



Basically all letters are small letters in the given key.

6. Press “OK” button to continue.

Activation of a different hardware (pressure plate) in the BalanceLab software

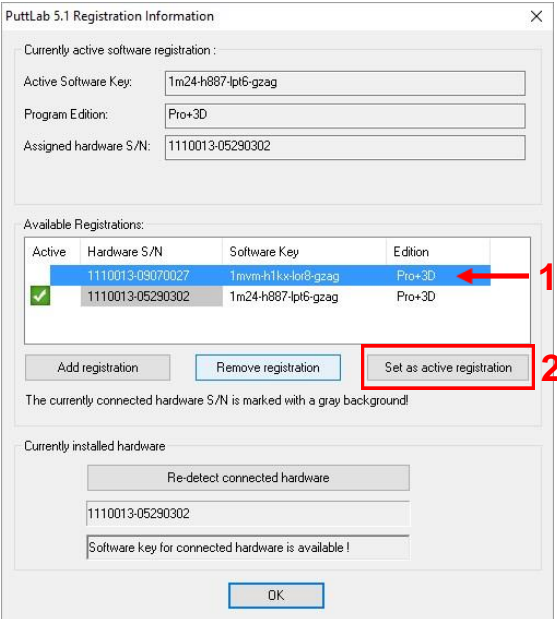

PuttLab 5.1 Registration Information

Currently active software registration :

Active Software Key: 1m24+h887-1pt6-gzag

Program Edition: Pro+3D

Assigned hardware S/N: 1110013-05290302

Available Registrations:

Active	Hardware S/N	Software Key	Edition
<input checked="" type="checkbox"/>	1110013-09070027	1mvm+h1kx-lor8-gzag	Pro+3D
<input type="checkbox"/>	1110013-05290302	1m24+h887-1pt6-gzag	Pro+3D

Add registration Remove registration Set as active registration

The currently connected hardware S/N is marked with a gray background!

Currently installed hardware

Re-detect connected hardware


1110013-05290302

Software key for connected hardware is available !

OK

☞ If the new key is accepted, you will now see the registration in the "Available Registrations" list.

7. Highlight the new registration by clicking the line with the mouse (should be blue highlighted after selection).
8. Press "Set as active registration" button.

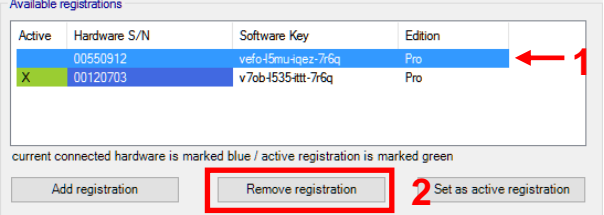

Available registrations

Active	Hardware S/N	Software Key	Edition
<input checked="" type="checkbox"/>	00550912	vefo+5mu-qez-76q	Pro
<input type="checkbox"/>	00120703	v7ob+535-ttt-76q	Pro

current connected hardware is marked blue / active registration is marked green

Add registration Remove registration Set as active registration

Delete registration for old unit

If the new hardware was a replacement for the old pressure plate and you'll only work with the new one, the old registration can be deleted:

- Select the line with the old Hardware/key in the "Available Registrations" list.
- Press "Remove registration" button.
- Press "OK" in the registration dialog and again start a measurement or training to check if the registration process was successful.



You cannot remove the only registration, as it would leave your software without valid license!



If you are still not successful with the registration, contact us under:
support@scienceandmotion.com